

Clinical Guideline

Air Volume Control

1-3 levels of operation to meet:

Orthodontics, Oral mucosal disease, Oral and Maxillofacial Surgery etc.

3-5 levels of operation to meet:

- 1) Periodontal ultrasound cleaning, Dental implant surgery, Crown and bridge tooth preparation etc.
- 2) Use an ultrasonic scaler, Ultrasonic bone knife machine, High-speed air turbine handpiece during treatment.

Cleaning and Disinfection

Cleaning:

1. Rinse the fog cover, Water filtration sponge with running tap water ($<40^{\circ}\text{C}$), and gently wipe the dirt visible on of the fog cover, Cleaning Water filtration sponge.
2. Wet cotton cloth with tap water, wipe the dirt on the surface of suction arm and the surface of the device.



- Caution: 1) When cleaning , avoid water entering the main unit, cause device damage.
2) Water filtration sponge can be used only after it has dried.

Disinfection:

1. Put the cotton cloth in a container containing 3% hydrogen peroxide solution or 75% medical alcohol to completely wet it, wipe fog cover, suction arm surface and main unit surface at least 3 times, make sure every location can be wiped.
2. Fill the watering can with 3% hydrogen peroxide solution or 75% medical alcohol, turn on the device, spray disinfectant into the suction arm, disinfect the inner arm.



- Caution: If you want to use other disinfectants, please use disinfectants that comply with local / national regulations and follow the regulations. (such as CE certification, FDA certification)